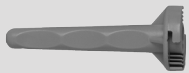
Vaginal Dilators Dilatateurs vaginaux Dilatadores vaginales

Five Graduated Sizes
Cinq dilatateurs de tailles croissantes
Cinco tamaños graduales

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Instructions for use | Mode d'emploi | Instructivo

Contents:
5 dilators, twist & lock
handle, lubricating gel,
instructions for use

About Amielle Comfort vaginal dilators

Amielle Comfort vaginal dilators have been designed for women experiencing vaginal discomfort and penetration problems from vaginismus, dyspareunia, gynecological surgery or trauma following childbirth. This guide was written in conjunction with psychosexual therapists.

How to use

Please read these instructions fully before using Amielle Comfort vaginal dilators for the first time.

Before you begin

Prior to first-time use, thoroughly wash dilators and handle in hot, soapy water. Rinse thoroughly and dry.

Privacy

Use the dilators in a private, comfortable setting where you will be undisturbed.

Positioning

Proper positioning is important to ensure optimal results. We recommend laying flat on your back, with knees bent and legs slightly separated. Alternatively, you may stand with one leg raised on a chair.

Progression

Start with the smallest dilator and once you can comfortably insert it, progress to the next size up. You may find when you first use the dilators you can only tolerate the tip (this is normal). With routine, daily use you should be able to gradually progress to full insertion, then larger sizes. When transitioning sizes, move from the smaller dilator to the next size in the same session, as vaginal muscles should already be relaxed and the transition should be easier. The time it takes to progress from size to size varies by individual. When you feel comfortable with inserting the larger dilators you may be ready to attempt penetrative sexual intercourse.

Inserting your dilator

Twist and lock the handle onto the dilator.

Apply water-based lubricant to the tip of the dilator and to your vaginal opening.

Since the vagina slants upwards and towards your back, the dilator should be angled upwards and in the direction of your back during insertion. Using controlled breathing (relaxing as much as possible), gently ease the dilator in an upwards and backwards direction as deeply as is comfortable. Leave the dilator in position for up to 5 minutes.

Duration & frequency

For best results, daily use is recommended. Leave the dilator in position for up to 5 minutes.

Cleaning

Amielle Comfort vaginal dilators and handle should be washed after each use. Wash in hot, soapy water. Rinse thoroughly and dry before storing in carrying bag. Ensure all traces of soap are removed to avoid irritation. Alternatively, you may immerse the dilators in boiling water.

Warnings & Precautions

These instructions are intended as a guide. Further instruction and advice may be obtained from your Healthcare Professional.

HEAD OFFICE
INTERNATIONAL SALES & MARKETING
Owen Mumford Ltd, Brook Hill, Woodstock
Oxfordshire OX20 1TU, United Kingdom
T: 01993 812021 F: 01993 813466
E: info@owenmumford.co.uk

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